

MEDORAH®

COLONOSCOPY PANTS

Instructions for Use

Medorah Meditek Pvt. Ltd.

www.medorah.com

MEDORAH® Colonoscopy Pants

Intended Use / Indications for use

MEDORAH® Colonoscopy Pants are intended for single-patient use in clinical settings to maintain hygiene and minimize the risk of contamination during colonoscopy procedures. These lightweight shorts feature a rear slit design to allow procedural access while maintaining patient comfort.

The pants also serve as a barrier to reduce the transfer of microorganisms, body fluids, viruses, and other contaminants between the patient and medical staff.

MEDORAH® Colonoscopy Pants is supplied non-sterile and is intended for single use only.

Device Description

MEDORAH® Colonoscopy Pants are made from a tri-layer, non-woven fabric that is water-repellent, breathable, and skin-friendly. The shorts-style fit with an elastic waistband ensures ease of wear and accommodates various body types. These pants support infection control protocols and promote efficient clinical workflows.

Specification: Standard/Universal size.



①	Colonoscopy Pants
---	-------------------

Contraindications

Not known

Potential Complications

Not known

Warnings

The entire User's Manual must be read thoroughly before using the product.

- Do not use if pouch is opened or damaged.
- Do not use the product for any other purpose other than its intended use.
- This product is designed for adult population only.
- This product is **non-sterile**.
- The product is intended for **single use only**. Do not reuse or reprocess, as this may increase the risk of infection, including cross-contamination between patients.
- Reuse may compromise the material integrity and hygienic function of the product.
- After use, dispose of the product and its packaging in accordance with hospital policy, local laws, and national biomedical waste disposal guidelines.

Procedure

- Open the packaging carefully and inspect the product for any signs of damage.
- Examine the pants, focusing on the waistband and edges, for any tears or defects.
- Dress the patient in the **MEDORAH® Colonoscopy Pants** before the procedure to avoid delays and ensure privacy.
- Ensure the opening with the attached cover is correctly positioned at the back. The opening allows access for the colonoscope while keeping the pants in place.
- Keep the pants in place throughout the procedure.
- Once the procedure is complete, carefully remove the pants.
- Discard the used pants as per hospital, local, and national biomedical waste disposal regulations.

Cautions:

1. Avoid contact with sharp or abrasive instruments or surfaces, which may cause damage or tearing of the material.
2. This product is for hygienic protection of the patient.

Storage

Store the device under defined temperature limit and avoid direct sunlight Follow the First-in, First-out rules and do not use the device beyond its expiry date.














Change of performance

If the device's performance is changed or the intended purpose is lost, further actions shall be taken according to the medical specialist's decision considering the clinical condition of patients.

Disposal of a used device

The device used must be disposed off according to the hospital, local and country regulations. Disposal is the responsibility of the user.

Explanation of symbols

	Non-Sterile		Keep dry
	Date of Manufacture		Use-By Date
	Catalogue Number		Do Not Reuse
	Consult Instructions for use		Caution
	Keep away from direct sunlight		Batch Code
	Do not use if package is damaged		Manufacturer
	Do Not Resterilize		

Warranty

Medorah Meditek Pvt. Ltd. warrants that this product has been manufactured by following appropriate procedures and reasonable care has been applied in designing and manufacturing of this device. This warranty is in lieu of and excludes all other warranties not expressly set forth herein, whether expressed or implied by operation of law or otherwise, including, but not limited to, any implied warranties of merchantability or fitness for a particular purpose. Handling, storage, cleaning and sterilization of this device as well as other factors relating to patient, diagnosis, treatment, surgical procedures, and other matters beyond Medorah Meditek Pvt. Ltd.'s control directly affect the device and the results obtained from its use. Medorah Meditek Pvt. Ltd.'s obligation under this warranty is limited to the repair or replacement of this device and Medorah Meditek Pvt. Ltd. shall not be liable for any incidental or consequential loss, damage, or expense directly or indirectly arising from the use of this device. Medorah Meditek Pvt. Ltd. neither assumes, nor authorizes any other person to assume for it, any other or additional liability or responsibility in connection with this device. Medorah Meditek Pvt. Ltd. assumes no liability with respect to devices reused, reprocessed or re-sterilized and makes no warranties, expressed or implied, including but not limited to merchantability or fitness for a particular purpose, with respect to such devices.



Mfd. By: Medorah Meditek Pvt. Ltd.

353, Udyog Vihar, Phase-IV, Gurugram, Haryana-122015 (INDIA)

Phone: 0124-4712471 | E-mail: info@medorah.com

www.medorah.com

License No. MFG/MD/2018/000026