

**MEDORAH®**

**Channel Cleaning Brush**

**Instructions for Use**

**Medorah Meditek Pvt. Ltd.**

**[www.medorah.com](http://www.medorah.com)**


# MEDORAH® Channel Cleaning Brush

## Intended Use


**MEDORAH® Channel Cleaning Brush** is used for cleaning the accessory channels and valves of endoscopes, ensuring safe and effective preparation of the device without causing channel damage.

## Device Description


MEDORAH® Channel Cleaning Brush is a specially designed device for manual cleaning of endoscope accessory channels and valves. Its flexible shaft and soft bristles allow thorough removal of debris and contaminants while minimizing the risk of channel damage, ensuring safe and effective preparation of the endoscope prior to disinfection or sterilization.

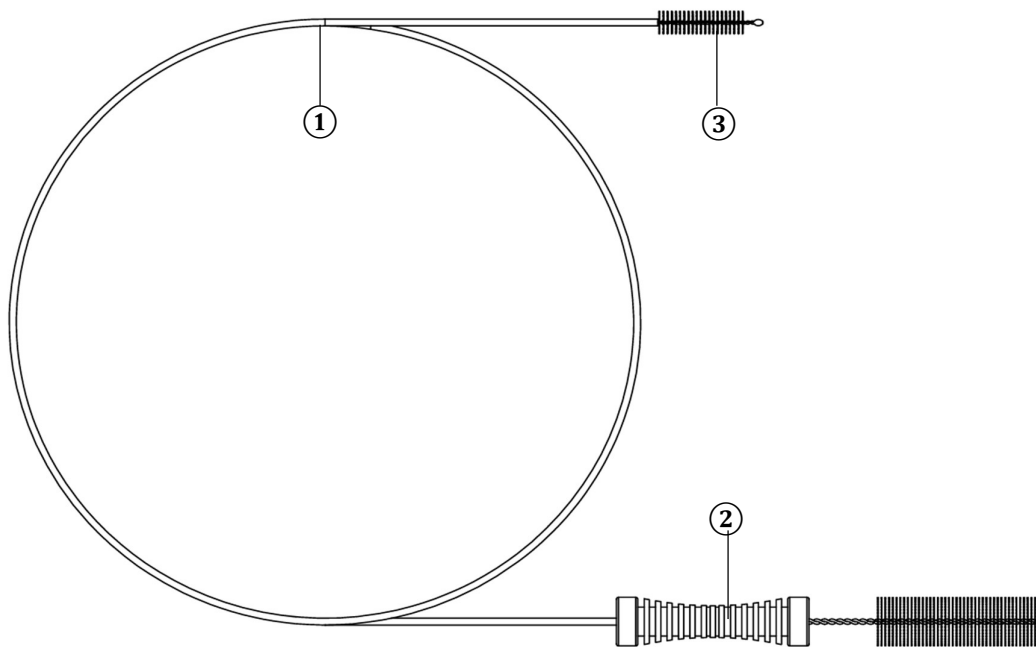

①	Catheter
②	Handle
③	Brush Bristle

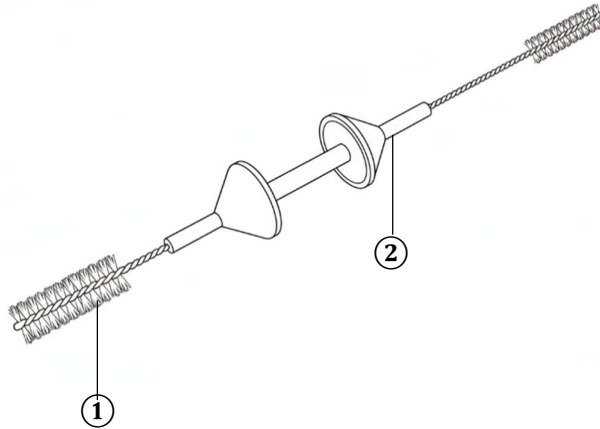

①	Brush Bristle
②	Handle

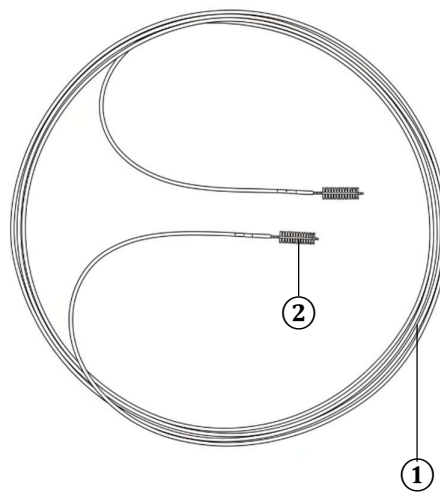

①	Catheter
②	Brush Bristle

### Contraindications

- There are no known contraindications.

### Complications

No complications are specific to the Cleaning Brush; risks are those inherent to endoscopic procedures.

## **Warnings**

- The device is supplied non-sterile.
- This device is intended for single use only. Reuse or reprocessing may damage the device, leading to malfunction, contamination, infection, or serious patient injury.
- If the device accidentally becomes dirty before treatment, it must be disposed of immediately. No cleaning agents may be applied.
- Do not use it if the pouch is opened or damaged.
- All components should be carefully checked for compatibility and integrity before use. Do not use a defective device. If defects occur, dispose of the device and replace it with a new one.
- Never use the product outside the recommended technical specifications (intended use).

## **Precautions**

- Ensure the diameter of the Cleaning Brush is compatible with the endoscope channel to achieve optimal cleaning results.
- Do not use excessive force when inserting or withdrawing the brush, as this may damage the endoscope channel.
- The Cleaning Brush is intended for use only by personnel trained in endoscope handling and reprocessing procedures.
- After use, the Cleaning Brush shall be treated as a potential biohazard and disposed of in accordance with hospital protocols, local regulations, and administrative guidelines.
- Always wear appropriate protective clothing (gloves, goggles, gown, etc.) while handling and using the Cleaning Brush.
- Read the Instructions for Use (IFU) carefully before use.

## **Procedure**

- Select the dual-end Cleaning Brush of appropriate diameter compatible with the endoscope channel.
- Gently insert the brush into the accessory channel of the endoscope until the channel is completely traversed.
- Move the brush back and forth several times to remove debris and ensure effective cleaning.
- Use the second brush end to clean additional channels or valves as required.
- After cleaning, carefully remove the brush from the channel and discard according to hospital and regulatory guidelines.












### Storage Conditions

Store the device under normal temperature and avoid direct sunlight. Follow the first-in, first-out rules.

### Disposal of a used device

Disposal of product and packaging is done in accordance with hospital, administrative, and/or local government policy by the user.

### Symbols used on the product label

	Manufacturing Date		Do Not Reuse
	Catalogue Number		Caution
	Consult the Instructions for use		Batch Code
	Keep away from direct sunlight		Manufacturer
	Do not use if the package is damaged		Keep dry
			Non-Sterile

### Warranty

Medorah Meditek Pvt. Ltd. warrants that this product has been manufactured by the appropriate procedures. This warranty is in lieu of and excludes all other warranties not expressly set forth herein which are beyond Medorah Meditek Pvt. Ltd. control such as warranties implied to the application of law, sales or specially purpose suitability after handling over, storage, cleaning and sterilization of this product as well as matters related to the patient, diagnosis, treatment, surgical procedures, and any other details. Medorah Meditek Pvt. Ltd. shall not be liable for any incidental, or consequential loss, damage or expense directly or indirectly arising from the use of this product other than the replacement of it. Medorah Meditek Pvt. Ltd. shall neither take any additional responsibility nor authorize such responsibility or duty to other person related to this product



**MEDORAH MEDITEK**

Mfd. By: Medorah Meditek Pvt. Ltd.

353, Udyog Vihar, Phase-IV, Gurugram, Haryana-122015 (INDIA)

Phone: 0124-4712471 | E-mail: [info@medorah.com](mailto:info@medorah.com)

[www.medorah.com](http://www.medorah.com)

License No. MFG/MD/2018/000026